

## Chairside Adjustment/Polishing for Zirconia



### STEP 1:

Start by gross adjusting and contouring using the coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.



### STEP 2:

Use the medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



### STEP 3:

Use the fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.

### Recommendation:



BRASSELER USA Finishing & Polishing Systems for Zirconia